

Air Device Instructions

Roll skin onto the inner gripper with the outer gripper in the up position.

Then secure the skin with the outer gripper by holding the tip of the device and inverting the outer gripper.

Inflate the device slowly **while pulling the device away from your body**. This is important so air is not forced directly into the urethra.

Taping over the urethra can be done but make sure every time you take off the device that you also remove the tape. Forgetting that the urethra is taped and trying to urinate can result in injury.

S clip attachment can be used for weight restoration.

Air pressure can be removed from the device by pinching the valve at the tip of the device, temporarily opening it to release air.

Disclaimer

The safety of this product is not guaranteed. Users must be careful to not injure themselves with the device, if you feel any pain - stop immediately.

DO NOT OVER INFLATE OR INFLATE FORCEFULLY AS THAT CAN RESULT IN INJURY